

## Description

Sheer White!® whitening films are the most comfortable, patient friendly whitening system available today. These films are designed to adhere to teeth for 1 to 2 hours to take advantage of the proprietary carbamide peroxide composition.

PLEASE NOTE: removal before 30 minutes can be done, but is more difficult, may tear and come off in pieces.




## Counter Indications

- Do not smoke during whitening treatment.
- Avoid contact with eyes. If contact occurs flush with copious amounts of water.
- Avoid direct contact of the active surface of the film with the gums and/or salivary flow.
- Not for use by women who are pregnant or nursing.
- Not for use by children under 18 years of age.
- Avoid contact with clothing.
- Do not swallow the film.
- If irritation (such as redness, swelling, soreness) of the gums or mouth occurs, discontinue use and consult a dentist.
- If you brux or grind your teeth, do not wear overnight.
- Do not use if allergic to peroxides like hydrogen peroxide.
- Do not use for more than 5 days per regimen.
- Whitening will not affect porcelain crowns and veneers tooth colored acrylic restorations or amalgam.

## Storage & Handling

- Keep the product refrigerated to maintain maximum shelf life.
- Do not store above 40°F (5°C). Do not freeze. Best use before date on label.
- Do not use if the packaging has been damaged, or if the seals are found to be broken or open.
- 20% initial concentration of carbamide peroxide. The concentration may change due to the nature of peroxide and this unique formulation.

### Shelf Life: 36 months


	Attention - Refer to the instructions for use and complete information.
	Oxidizer - Avoid contact with reactive substances.
	Storage Temperature - Store product at the indicated temperature.



1 The whitening side of the film faces UP. Grasp film at end and remove as shown.



2 Center film and place against teeth at gum line.

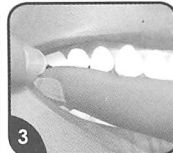

3 Pull cheek away and push film onto back teeth.



4 Fold film to back side of teeth. Press firmly on all surfaces.



Wear  
1-2 hours



6 To remove, grasp one corner and pull down away from front of teeth, then push film away from back of teeth.



7 Brush or rinse away any remaining gel. Rinse to sink.